Instructions for Use

Ambu[®] UltraSeal

Disposable Anesthesia Face Mask
Non-sterile, Non-conductive, with/without check valve
Single Patient Use Face Mask

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Ambu® is a registered trademark of Ambu A/S, Denmark.
Ambu is certified according to ISO 13485.

Contents

	Page
English (Instructions for use).....	4
България (Ръководство за употреба)	5
Česky (Návod k použití)	6
Dansk (Brugsanvisning).....	7
Deutsch (Bedienungsanleitung)	8
Ελληνικά (Οδηγίες χρήσης).....	9
Español (Instrucciones de uso).....	10
Eesti (Kasutusjuhised).....	11
Suomi (Käyttöohje).....	12
Français (Mode d'emploi).....	13
Hrvatski (Upute za upotrebu).....	14
Magyar (Használati útmutató).....	15
Italiano (Istruzioni per l'uso).....	16
日本語 (使用法)	17
Lietuviškai (Naudojimo instrukcijos).....	18
Latviski (Lietošanas instrukcija)	19
Nederlands (Gebruiksaanwijzing)	20
Norsk (Bruksanvisning)	21
Polski (Instrukcja użytkowania).....	22
Português (Instruções de utilização)	23
Română (Instrucțiuni de utilizare)	24
Русский (Инструкция по применению).....	25
Slovenčina (Návod na použitie)	26
Slovenščina (Navodila za uporabo).....	27
Svenska (Bruksanvisning).....	28
Türkiye (Kullanım kılavuzu).....	29
中文 (使用说明)	30

Intended Use

The Ambu UltraSeal is a single patient use disposable Face Mask intended for oxygenating and ventilating the airway or to direct anesthetic gases to the upper airway.

The device is for use in hospital environment, typically emergency room (ER) and operating room (OR).

For use by medical professionals in the field of anesthesiology, respiratory care and users educated in performing CPR.

Instructions for Use

1. Remove mask from package and inspect for proper inflation.
2. Inspect the cuff for any damage or leakage.
3. Remove and discard the four-prong hook ring if not required (if apply).
4. If needed, use syringe to add/remove air to achieve desired cuff pressure (mask with check valve only).
5. Apply the mask firmly to the patient's face to achieve a tight seal. Hold the mask tight against the face while maintaining the correct head tilt to keep the airway open.
6. If deemed necessary, the surface of the product can be manually cleaned with water and a mild detergent. Rinse thoroughly with clean water. Allow to dry, and check that the mask is fully operational before re-use on the same patient. If not fully operational, the mask shall be discarded.
7. Discard mask after use according to local guidelines of medical waste.

WARNINGS

- For single patient use only.
- Use on other patients can cause cross infection.
- The design and material of the device are not compatible with automatic cleaning or sterilization processes, as it may leave harmful residues or cause malfunction of the device.

CAUTION

- Do not attempt to attach the mask to any connection or adaptor not compatibly sized.

Storage Temperature range: Tested at -40 °C and +70 °C (-40 °F and +158 °F)
according to EN ISO 10651-4 and EN 1789

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