Ambu® Open Cuff Silicone Face Mask

EN

Intended Use
The Ambu® Open Cuff Silicone Face Mask is a reusable face mask intended for use with manual resuscitators, automatic resuscitators and ventilators.

For use by CPR trained personnel only.

Instructions for Use

Remove mask from package and inspect. Apply the mask firmly to the patient's face to achieve a tight seal. Hold the mask tight against the face while maintaining the correct head tilt to keep the airway open. Clean and sterilise the face mask according to instructions.

Materials:

Silicone Rubber

Cleaning-disinfection-sterilisation - Reusable Item ONLY

To ensure safety of all kinds, please clean and sterilize

- a. Between patients.
- b. Whenever the product becomes contaminated.
- c. Every 24 hours of use with the same patient.

Carry out cleaning and sterilization as follows:

- Wash the components thoroughly in warm water containing a mild detergent. Ensure that the detergent is compatible with the component materials.
- Rinse the components thoroughly in warm water to remove all traces of detergent or fluid.
- Sterilise or disinfect the components using one of the following methods:
 - a. Autoclave up to 134 °C or 273 °F.
 - b. Most common disinfecting solutions that are used for equipment coming into contact with the patient.
 - Rinse thoroughly with water after disinfection.
 - Dry all components thoroughly after disinfection.
 - Inspect all components for wear and tear or damage, and replace them as necessary.
- Then package appropriately to protect the device from soil and contamination.

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