



Ambu® Blue Sensor NEO X User Guide


Placement

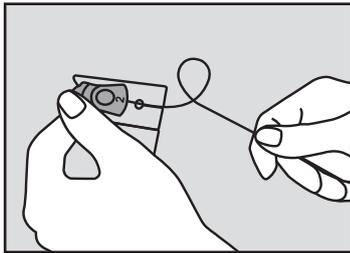


Straighten the lead wire.

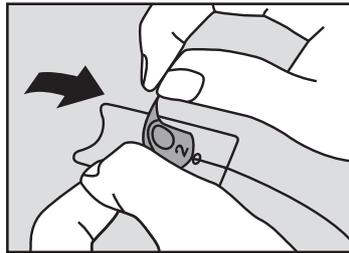


Remove the electrode from the plastic liner by pulling it towards you – using the white tab "1".

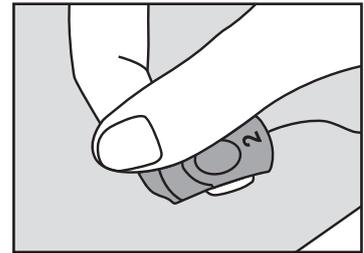


Place the electrode on the patient's skin.

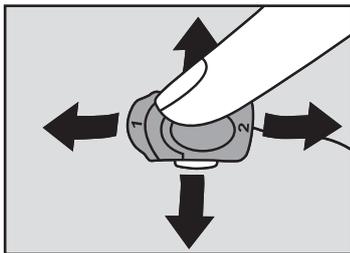

Gently apply pressure with your finger on each of the four leaves around the edges of the electrode.

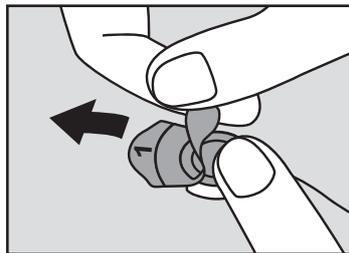


Remove the tab "2" by pulling it upwards, parallel to the skin. At the same time, hold the electrode in place with your other hand.

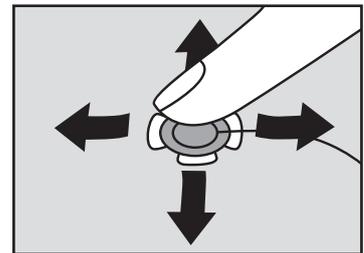


Gently apply pressure with your finger on each of the four leaves around the edges of the electrode.


Removal

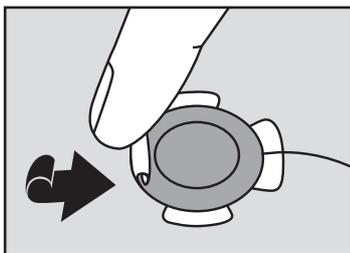

Gently roll the leaves of the electrode towards the centre.

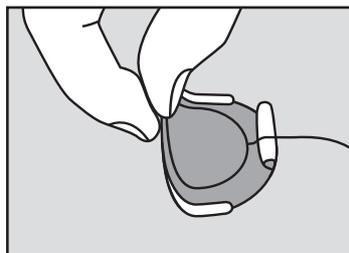

Lift up the electrode with your finger between the adhesive and the patient's skin.

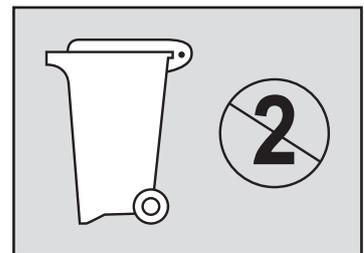

Dispose the electrodes – the NEO X electrodes are not repositionable.