

Quick Guide Ambu® Manikin Management Module

Quick Course: Start several manikins at the same time



This guide explains the new functionality "Quick Course" which you can use to start quickly a new session on several manikins simultaneously when they are connected to each other.

Please consider the quick guide "Create a wireless access point" in which you will find out how to connect a maximum of 5 client devices to 1 host device.

Connect the training devices and control them by the host manikin.

1. Access „Controlled session“



Start the host training device and connect your end device to it. Enter **http://Ambu.login** or **192.168.1.1** into the address bar of your browser and select „Controlled session“.

2. Manage Quick Course

Now click on the menu item "Quick Course".


In this view you can see the settings for the session and in the lower display all available manikins with their name and IP address.

The screenshot shows the "Quick Course" configuration interface. On the left is a navigation menu with items: New Session, Session View, Surveillance View, Team Debriefing, Quick Course (highlighted), Training Course, Database, and Configuration. The main area is titled "Quick Course" and contains the following settings:

- Date:** 2018-10-23
- Time:** 08 : 30
- Scenario:** Default scenario
- Algorithm:** ERC 2015
- Duration:** 10 minutes
- Training mode:** Two person CPR
- Instructor:** Instructor 1
- Manikin:** All Selected

Below the settings is a large green play button. At the bottom, there is a table for "Connected manikins":

	Name	IP address
<input checked="" type="checkbox"/>	AmbuMan_Host	192.168.1.1
<input checked="" type="checkbox"/>	AmbuMan_Client_1	192.168.1.21
<input checked="" type="checkbox"/>	AmbuMan_Client_2	192.168.1.31

Select the parameters for your session and click on the "Play" button  to open a new training session.

By default, all connected manikins will be started simultaneously which can be recognized on the greyed out check boxes.

Continuation: Manage Quick Course

Quick Course

Date: 2018-10-23 Time: 08:30 Scenario: Default scenario

Algorithm: ERC 2015 Duration: 1 minute Training mode: Two person CPR


Instructor: Instructor 1

Manikin: All Selected

Connected manikins

	Name	IP address
<input checked="" type="checkbox"/>	AmbuMan_Host	192.168.1.1
<input type="checkbox"/>	AmbuMan_Client_1	192.168.1.21
<input checked="" type="checkbox"/>	AmbuMan_Client_2	192.168.1.31

If you want to choose between all connected manikins, change the „Manikin“ setting from „All“ to „Selected“ and click on the check boxes of the devices you want to start.

After that, click on the „Play“ button  to open a new training session.

Note: The Quick Course is started standardly with the default trainees “Trainee 1” and “Trainee 2” and can't be changed.

3. Surveillance view

Surveillance View

AmbuMan_Host

AmbuMan_Client_2


Ventilation: Trainee 1 (Trainee 1)

Compression: Trainee 2 (Trainee 2)

Now you see all connected and selected devices in the surveillance view where you can start the session on each manikin at the same time or independently.

By default, all manikins are selected to start them at once. For this, click on the “Play“ button .

Of course, you can also select the manikins you want to start or control independently by clicking on the grey button in the upper display with the device name and power source.

To end a session, go to the menu item “Session View”, select the device with the grey button and click on the “Exit“ button .